ENGLISH INSTRUCTIONS FOR USE

I. INTRODUCTION

KATANA Cleaner is a non-abrasive dental cleaning agent indicated for cleaning bonding surfaces of prosthetic restorations after intraoral try-in as well as dentin, enamel, root canals, tooth abutments and implant abutment.

II. INDICATIONS FOR USE

KATANA Cleaner is indicated for the following uses:

- Cleaning of contaminated glass ceramic, zirconia, resin restorations, metal restorations and fiber post after intraoral try-in.
- Cleaning of contaminated prepared tooth (cavity, root canal and tooth abutment), and implant abutment.

III. CONTRAINDICATIONS

Patients with a history of hypersensitivity to methacrylates monomers.

Patients known to be allergic to any of the ingredients contained in this product.

IV. PRECAUTIONS

1. Safety precautions

If the patient demonstrates an allergic reaction, discontinue use of the product and seek medical attention.

If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching or numbness, discontinue use of the product and seek medical attention.

Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomers or any other components.

Exercise caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, wear eye protection with a towel to protect them in the event of splashing material.

If the product comes in contact with human body tissues, take the following actions:

- If the product gets in the eye: Immediately wash the eye with copious amounts of water and consult a physician.

- If the product comes in contact with the skin or oral mucosa: Immediately wipe the area with a cotton pellet or a gauze pad moistened with alcohol, and rinse with copious amount of water.

Exercise caution to prevent the patient from accidentally swallowing the product.

Dispose of this product as a medical waste to prevent infection.

2. Handling and manipulation precautions

This product must not be used for any purposes other than specified in [I]. INDICATIONS FOR USE.

The use of this product is restricted to dental professionals.

Use a pulp capping agent in a cavity close to the pulp in the event of accidental pulp exposure.

Use a rubber dam to prevent contamination and to control moisture.

Do not mix the product with any other dental materials.

The container should be tightly capped immediately after use. If the liquid cannot be dispensed easily from the nozzle, do not forcibly dispense from the pluggist container.

Be careful not to contaminate the surface with saliva or blood. If the treated surface is contaminated, re-treat using the product.

Use the product within 10 minutes after dispensing.

Wipe the nozzle with a gauze after using to avoid solidification of the liquid.

Do not shake the bottle to avoid bubbles formation and to avoid contamination of the nozzle.

Although bubbles may be contained within the dispensed liquid, it will be resolved during application and will not affect the performance of the product.

3. Storage precautions

This product must not be used after the expiration date indicated on the package.

The product must be stored at 2-25°C/36-77°F when not in use.

The product must be kept away from direct sunlight.

The product must be stored in a proper place where only dental practitioners have access.

If the product is in refrigerated when not in use, it must be brought to room temperature within than 15 minutes before using; otherwise, an excessive amount of liquid may be dispensed or the liquid may coze after use. If this occurs, wipe away the excess with a gauze before using.

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