

ANTIMICROBIAL* HYDROGEL

Vetericyn Plus® VF Antimicrobial Hydrogel is designed to adhere to the site of application, allowing the advanced hypochlorous formula to penetrate the wound bed to add and maintain moisture balance. This unique hydrogel does not cause dermal irritation and is safe for use in sensitive areas. Vetericyn Plus Antimicrobial Hydrogel is an excellent post-surgical wound care solution.

A moist wound and skin environment facilitates autolytic debridement and is beneficial to wound management and the wound healing process. Our hydrogel has an extensive safety profile, won't harm healthy tissues, and works in tandem with Vetericyn Plus Wound & Skin Care Liquid to deliver a powerful combination for wound care.



16 oz Hydrogel Wound & Skin Care

Product #: 2040

UPC #: 852009002222

Size: 16 oz

Bottle Dimensions:
8.25" x 2.75"

Bottle Weight (lbs): 1.20

Bottles per Case: 6

Case Weight (lbs): 7.5



8 oz Hydrogel Wound & Skin Care

Product #: 2041

UPC #: 852009002239

Size: 8 oz

Bottle Dimensions:
6.5" x 2.25"

Bottle Weight (lbs): 0.6

Bottles per Case: 6

Case Weight (lbs): 3.9



4 oz Hydrogel Wound & Skin Care

Product #: 2043

UPC #: 852009002277

Size: 4 oz

Bottle Dimensions:
5.5" x 1.75"

Bottle Weight (lbs): 0.3

Bottles per Case: 6

Case Weight (lbs): 2.05



2 oz Hydrogel Wound & Skin Care

Product #: 2042

UPC #: 818582011358

Size: 2 oz

Bottle Dimensions:
4.5" x 1.4"

Bottle Weight (lbs): 0.15

Bottles per Case: 12

Case Weight (lbs): 2.4

*Vetericyn Plus Antimicrobial Hydrogel contains hypochlorous acid, a known antimicrobial, which serves as a preservative to inhibit the growth of microorganisms in the hydrogel.

DIRECTIONS FOR USE - HYDROGEL

1. Shake well before use
2. Remove excess hair from wound area
3. Clip if necessary
4. Adjust spray nozzle and saturate the affected area
5. If required, fill cavity with appropriate wound filler and then cover with gauze or other covering and secure into place
6. Reapply as required to ensure adequate moisture balance
7. Hydrogel is designed to stick to the site of application and evaporate slowly; no need to wipe away film or residue

Safe for use around eyes, ears, nose and mouth. Faster healing is achieved when wounds are cleaned and maintained in a moist environment. For best results, first clean affected area with Vetericyn Plus VF Wound & Skin Care Liquid to ensure deep dermal cleansing and proper wound bed preparation

IN-CLINIC USES

Wound Care Applications

- + Acute and chronic wounds
- + Partial and full-thickness wounds
- + Abrasions and abscesses
- + 1st and 2nd degree burns
- + Post-surgical wounds
- + Grafted mesh, pinch, punch, strip, and donor sites
- + Stage I-IV skin ulcerations
- + Minor lacerations and cuts
- + Moistening and lubricating wound dressings
- + Reducing wound odors

Dermatologic Applications

- + Skin rashes
- + Allergy sites
- + Fecal or urine contact skin irritations
- + Feather-picking-related irritations
- + Hot spots

PRESCRIPTION USE FOR CARE AT HOME

- + Post-procedure incision sites
- + Dermal irritation from friction sites and skin folds
- + Moistening and lubricating during dressing changes

