

USER MANUAL

Steel 4-wheeled rollator

Product code: TGR-R RS 881



1. Characteristics

The folding 4-wheeled rollator is made of a powder-coated steel frame. It is equipped with parking brakes, a seat with a removable tray, and a basket. The product comes with solid rubber wheels 8" in diameter and height-adjustable handles with anatomical handgrips.

The set includes: frame with seat, push handles with brakes, basket, tray, locking knobs (2 pcs).

Elements of the rollator:

- A. Push handle
- B. Parking brake
- C. Locking knob
- D. Frame folding mechanism
- E. Transverse hinge
- F. Seat with tray
- G. Crutch holder
- H. Upper adjustment of brake
- I. Lower adjustment of brake
- J. Basket



2. Application

The rollator is designed for patients with reduced mobility who need stable support and assistance while walking. It is also recommended for the elderly as an aid in maintaining balance, improving walking comfort, and ensuring protection against falls. The use of the rollator has a positive effect on restoring activity in people with mobility impairments and difficulties standing up. It provides improved stabilization, support, and safety while walking.

3. Assembly

Unfolding and folding the rollator

- First, insert the push handles into the mounting openings at the appropriate height by tightening the locking knobs.
- To unfold the rollator, press the crossbar which is located under the seat, so that the hinges on the outer sides of the rollator are completely unfolded.
- Hang the basket on the front frame, beneath the seat.
- Mount the tray onto the seat, ensuring that the tray catches fit securely into the mounting openings.
- Remove plastic wrap from the wheels.
- To fold the rollator, engage the brake by pulling the handle firmly downwards. Then press the frame-folding lock located on the right while simultaneously pulling the crossbar upwards. The basket and tray can be removed if necessary.

NOTE! Be careful while folding the rollator due to the risk of smashing fingers in hinges.

BRAKES

- To stop the rollator, pull the brake handle upwards and hold.
- While stop or using the seat, lock the parking brakes by pushing down the brake handle.
- To release the brake, pull the brake handle upwards.

HEIGHT ADJUSTMENT OF HANDLES

To adjust the height of the handles, unscrew the locking knobs located on the outer frame, select appropriate height and tighten them afterwards.

NOTE! The handles must be set on the same level.

ADJUSTMENT OF BRAKES

Adjustment of upper brake

- Unscrew the nut of the cable, pull it up to the appropriate length and lock it by tightening the locking nut.

Adjustment of lower brake

- Loosen the mounting screw with the use of open-end wrench, pull the black brake cable to adjust the tension of the cable, and then, tighten the mounting screw.

4. Technical data

Overall width	57 cm
Overall depth	65 cm
Seat dimensions	38 x 16 cm
Seat height from the ground	62 cm
Handles height adjustment	79-91 cm
Weight	10,8 kg
Maximum capacity	135 kg
Maximum capacity of the basket	12 kg
Frame colour	dark blue

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Cleaning and maintenance

- Clean the frame with damp cloth and dry thoroughly.
- Do not use strong polishing agents due to the damage of protecting coatings.
- Clean the dirt from moving parts of the rollator regularly (hinges, clamping screws, locks).

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: **www.timago.com**. Thank you for choosing Timago!