

INSTRUCTIONS FOR USE

Aluminium 4-wheeled rollator with footrest

Product code: TGR-R RA 888

Timago

1. Characteristics


Adjustable 4-wheeled rollator is made of aluminium frame which is coated with paint with metallic lustre. The rollator consists of upholstered seat, soft backrest, foldable footrest and detachable basket. The product is also supplied with height adjustable push handles with anatomic handgrips and brakes with locking mechanism. The rollator is equipped with wheels with narrow cross section 8" in diameter.

The set includes:

- Two push handles with locking mechanism
- Frame with seat
- Backrest
- Basket
- Locking knobs with washers – 2 pcs

Elements of the rollator:

- a. push handle
- b. brake
- c. locking knob
- d. backrest
- e. fixing of backrest
- f. basket
- g. hinge
- h. foldable footrest
- i. upper adjustment of brake
- j. lower adjustment of brake

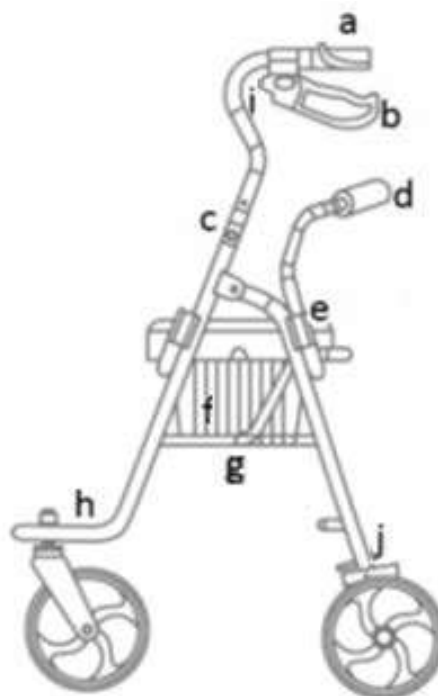

2. Application

The rollator is designed for patients with reduced mobility who need stable support and help while walking. It is also recommended to the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the rollator affects positively on the return to activity of the people with locomotion and tilting to erect position problems. It guarantees better stabilization, support and security while walking.

3. Assembly

Unfolding and folding the rollator

- To unfold the rollator, press the side crossbar, which is placed under the seat, until the hinges are completely unfolded. Then, fix the handles into the mounting openings and select appropriate height by installing and tightening the locking knobs. Brake cables should be placed at the front.
- Fix the basket under the seat and the seat on unfolded frame.
- Fix the backrest into the mounting openings at the front of the rollator until the pins are locked.
- Remove plastic casing from wheels.
- Unfold the backrest while temporary rest. Fold the backrest while standing up from the rollator.
- To fold the rollator for transport, lift the seat, take out the basket and pull upwards the strap which joins the side crossbar with frame until the frame is completely folded.

Brakes

- To stop the rollator, pull the brake handle upwards and hold.
- While stop or using the seat, lock the brake with locking mechanism by pushing down the brake handle.
- To release the brake, pull the brake handle upwards.

Height adjustment of handles

- To adjust the height of the handles, unscrew the locking knobs which are located on the outer side of the frame, select appropriate height and tighten the locking knobs.

Adjustment of brakes

Adjustment of upper brake

- Unscrew the nut of the cable, pull it up to the appropriate length and lock it by tightening the locking nut.

Adjustment of lower brake

- Loosen the mounting screw with the use of open-end wrench, pull the black brake cable to adjust the tension of the cable, and then, tighten the mounting screw.

4. Technical data

Overall width	61 cm
Overall depth	80 cm
Seat dimensions	35 cm x 32 cm
Seat height from the ground	55 cm
Handles height adjustment	78-88 cm
Weight	8.2 kg
Maximum capacity	110 kg
Maximum capacity of the basket	5 kg

5. Notes

Timago International Group does not bear responsibility of improper use of the rollator, failure to comply safety regulations and misuse.

6. Maintenance

Reviews

Make visual inspection of all the joining elements (clamps, hinges) of the rollator at least once every three months including:

- Brakes

- Mounting elements (pins, nuts, knobs)
- Height adjustment of handles
- Stability and look of the frame

Cleaning

- Clean the frame with damp cloth and dry thoroughly.
- Do not use strong polishing agents due to the damage of protecting coatings.
- Clean the dirt from moving parts of the rollator regularly (hinges, clamping screws, locks).

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction



The manufacturer checked for compliance with essential medical devices requirements.

Our complete line of products is available from our distributors.
Learn more at: www.timago.com. Thank you for choosing Timago!



Timago International Group
Spółka z o.o. i Spółka – Spółka komandytowa
ul. Karpacka 24/12
43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00
F.: +48 33 499 50 11
E.: info@timago.com



Rev. II, date 20.05.2021