
Instructions

Saliva Test for Cannabis (THC)

This rapid test is used to detect the presence of THC (the active substance in cannabis) in saliva. It is a qualitative test, meaning it shows whether THC is above or below a certain cut-off level – in this case, 50 ng/mL. The result indicates recent cannabis use, with a detection window of up to 14 hours.

How to perform the test:

1. Preparation: The user must not eat, drink, chew gum, or smoke for 10 minutes before the test.
2. Saliva Collection:
 - Remove the cap from the test stick.
 - Place the tip of the tongue against the teeth and gums to stimulate saliva.
 - Insert the sponge end of the test stick into the mouth and rub it against the gums on both sides (10–15 times).
 - Then place the sponge under the tongue until liquid starts moving in the test window (approx. 60 seconds).
3. Reading:
 - Place the test on a flat surface.
 - Read the result within 3–10 minutes.
 - Do not read after 10 minutes.

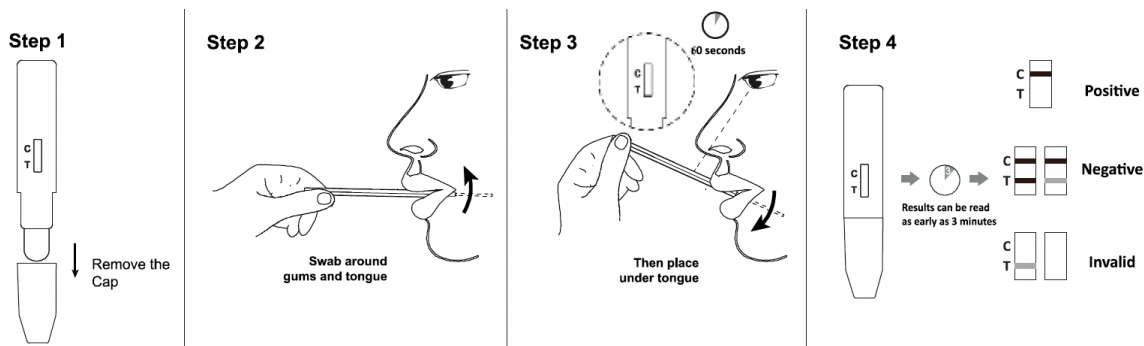Interpreting the result:

NEGATIVE: Two colored lines appear (one in the control region C, one in the test region T). This means the THC level is below 50 ng/mL.

POSITIVE: Only one line appears in the control region C. No line in the test region T. This indicates that the THC level is above 50 ng/mL.

INVALID: No line appears in the control region C → the test failed. Repeat the test with a new test stick.

Note: Any shade of color in the test line should be considered a negative result.

Reliability:

The test has been compared to lab methods such as GC/MS and LC/MS:

- Sensitivity (true positives): 96.2%
- Specificity (true negatives): 97.1%
- Overall agreement with lab results: 96.8%

This means the test is highly reliable, but note:

- A positive rapid test should be confirmed with a lab test for legal or medical purposes.
- A negative result does not always exclude THC presence – low dose or incorrect sampling may affect results.

Storage:

Store the test in unopened packaging at 2–30 °C. Do not freeze. Do not use after the expiration date.

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