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Manual Milk & Colostrum Pump

About the product and its use

The VetProvide Manual Milk Pump is a handheld tool designed for collecting a mare's milk when a foal is unable to nurse independently, when colostrum needs to be harvested for storage, or when a mare requires immediate relief from milk congestion. This pump serves as a practical aid for acute, short-term care in the stable.

Common situations requiring a milk pump:

- The foal is premature, weak, or physically unable to nurse.
- The mare exhibits behavioural issues or refuses to let the foal nurse.
- Colostrum needs to be harvested and frozen as an emergency reserve.
- The mare is experiencing painful milk congestion or engorgement.

Clinical note: The importance of colostrum

Colostrum (the mare's first milk) is exceptionally rich in antibodies that establish the foal's immune system. A newborn foal must receive high-quality colostrum within the **first 2 hours of life** to ensure adequate immunity and survival. Contact your vet immediately if the foal has not nursed within 2 hours of birth.

WHAT YOU WILL NEED

- Clean hands
- Lukewarm water and a clean cloth for the udder
- A sterile or thoroughly cleaned container for milk collection
- Standard dishwashing liquid and a dedicated clean brush for cleaning the pump
- Clean paper towels or a clean drying rack

BEFORE YOU BEGIN

- Wash your hands thoroughly with soap and warm water for at least 20 seconds.
- Ensure all pump components are completely clean and dry before assembly.
- Choose a quiet, familiar, and safe environment to minimise stress for the mare.
- Ensure the mare is safely and securely restrained, allowing you to reach the udder comfortably.

STEP-BY-STEP INSTRUCTIONS

1 Prepare the udder

Gently wash the udder and teats with lukewarm water. Pat completely dry with a clean paper towel. Lightly massage the udder for 30 to 60 seconds using calm, even strokes to stimulate milk let-down and help the mare relax.

2 Assemble and inspect the pump

Verify that all components are correctly fitted together. Check for any physical cracks or wear. Never use a damaged or cracked pump. Replace worn components immediately to maintain proper suction and hygiene.

3 Position the funnel

Place the funnel cup opening snugly and squarely over the teat to create a secure, airtight seal without causing discomfort. *Tip: If the mare is restless, have an assistant securely hold her head and calm her while you focus on pumping.*

4 Pump with a steady rhythm

Depress the handle gently and evenly in a calm, rhythmic motion. Start slowly to mimic a foal and stimulate initial flow, then steadily increase the pace once milk begins to draw. Do not squeeze the handle with excessive force, as this can cause tissue bruising or pain on the teat. If harvesting from both sides, alternate periodically between teats.

5 Collect the milk

Hold the collection bottle stable throughout the process. Stop pumping once you have collected the required volume or when the milk flow visibly diminishes. If feeding the foal immediately, do so right away using clean equipment. Otherwise, follow the guidance in the "Important hygiene and storage tips" section below.

6 Clean immediately after use

Disassemble all components that came into contact with milk.

- **Rinse first with cold water:** This is a critical step to prevent milk proteins from bonding onto the plastic surfaces. Do not use hot water first.
- **Wash with warm water:** Wash thoroughly using warm water, standard dishwashing liquid, and a clean brush.

- **Rinse and air-dry:** Rinse completely with clean water to remove all soap residue. Place components on a clean surface or rack to air-dry fully. Do not dry with cloth towels as they can harbour hidden bacteria. Store in a clean, dust-free space.

IMPORTANT HYGIENE AND STORAGE TIPS

- **Freezing colostrum:** If storing colostrum, freeze it within one hour of collection in a clean, airtight, date-labelled container.
- **Thawing rules:** Never thaw colostrum in a microwave. High heat permanently destroys the antibodies. Always thaw slowly by submerging the container in lukewarm water.
- **No shared equipment:** Do not use the pump across different animals without a thorough disinfection process between uses.
- **Soap selection:** Use standard dishwashing liquid. Avoid harsh antibacterial liquids that can degrade the pump materials or leave chemical residues.

When should you contact a vet?

Newborn foals and post-foaling mares are clinically fragile. Contact your vet immediately if:

- The newborn foal has not successfully stood up and nursed within 2 hours of birth.
- The foal appears weak, dull, or sluggish, lacks a natural sucking reflex, or fails to seek the udder.
- The mare's udder feels hot, hard, abnormally swollen, or is severely painful to the touch.
- The expressed milk looks abnormal, for example watery, clumpy, stringy, blood-stained, or deeply discoloured.
- The mare develops a fever, acts depressed, or walks with a stiff, guarded hindlimb gait.
- The mare fiercely resists gentle pumping attempts despite a calm environment.

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