

Dear customer,

The RUCK MOBILE SYSTEM technical unit is the practical bag for all technical products for professional mobile foot care. The timeless professional exterior of the bag is made of sturdy, water-repellent tarpaulin for trucks. The insert with integrated rotatable trays made of shock-proof ABS plastic is hygienic and easy to clean. Unpack – connect – and you're ready! By concentrating on the essentials and dispensing with unnecessary storage space, we have kept the weight of the technical unit to a minimum. Thanks to the compact dimensions, it can be conveniently placed wherever it is needed.

The technical unit system has space for all PODOLOG foot care devices, the professional magnifier lamp PODOLOG CIRCLE S mobile, the office folder set and a 3-metre extension cable.

Please read the following operating instructions carefully all the way through before commissioning the device. If you have any further questions, please do not hesitate to contact us.

We sincerely thank you for the trust you have placed in our company and extend our best wishes for satisfaction in your daily work with your new technical unit.

Kind regards, HELLMUT RUCK GmbH

THE ENTIRE RANGE OF TECHNOLOGY IS INCLUDED



1 The technical unit
Bag and rucksack

EVERYTHING IS THERE FOR YOU – JUST IN CASE



2 The work units
S and M

SUPPLIES ARE ALWAYS AT HAND



3 The supplies unit

TABLE OF CONTENTS

GENERAL INFORMATION	PAGE
	5
PRODUCT DESCRIPTION	6
INSERTING THE FOOT CARE DEVICE INTO THE TECHNICAL UNIT	8
MOUNTING THE MAGNIFIER LAMP AND HANDPIECE HOLDER	14
CLEANING AND DISINFECTING THE TECHNICAL UNIT	15
WARRANTY	18
TECHNICAL DATA	19



Easy to pack,
VIDEO at www.hellmut-ruck.de/special/mobilsystem

DECLARATION RELATING TO THESE OPERATING INSTRUCTIONS

Hellmut Ruck GmbH reserves all rights to modify and supplement these operating instructions. Hellmut Ruck GmbH has taken great care to keep these instructions free of inaccuracies and omissions. Please contact us if you find mistakes or inaccuracies. However, Hellmut Ruck GmbH assumes no liability for mistakes in these operating instructions or consequential damage resulting from the provision of these operating instructions or their application when using the product.

These operating instructions must not be disseminated or supplemented without the express approval of Hellmut Ruck GmbH.

SYMBOLS USED



This symbol identifies a potential point of danger for people or for the device itself. It must be taken note of.



This symbol identifies useful advice. Additional information about the product and how to use it is provided here.



PACKAGING

The packaging has been designed to protect the product while in transit. The packaging materials used have been selected with due regard for environmental issues and can be recycled.

THE ENTIRE RANGE OF TECHNOLOGY IS INCLUDED FOR YOU - JUST IN CASE

EVERYTHING IS THERE SUPPLIES

ARE ALWAYS AT HAND



1 The technical units
Bag and rucksack

2 The work units
S and M

3 The supplies unit



19

18



01



02

1.1

03


04

05

06



07

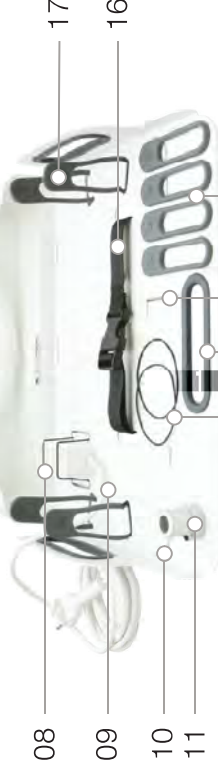

08

09

10

11



12

13

14

15

16


17

- 01 PODOLOG CIRCLE S MOBILE (optional)
- 1.1 JOINT CAP
- 02 ROTATABLE INSTRUMENT TRAYS
- 03 ROTATABLE HANDPIECE HOLDER
- 04 HANDPIECE STORAGE LOCATION
- 05 DEVICE COVER
- 06 BASE PLATE
- 07 SECURING THE ROTATABLE INSTRUMENT TRAYS
- 08 MULTISOCKET OUTLET SECURED WITH O-RING
- 09 CABLE FIXING
- 10 GUIDE GROOVE FOR DEVICE COVER
- 11 LIGHT BRACKET WITH HEADLESS SCREW
- 12 O-RING (1 x for the power supply of the PODOLOG CIRCLE S, 1 x replacement)
- 13 SILICONE GRIP INSERT
- 14 ALLEN KEY FOR LIGHT BRACKETS
- 15 4 x ELASTIC STRAP (replacement)
- 16 BELT FASTENER FOR FOOT CARE DEVICE
- 17 ELASTIC STRAPS FOR DEVICE COVER
- 18 TECHNICAL UNIT BAG
- 19 TECHNICAL UNIT RUCKSACK


INSERTING THE FOOT CARE DEVICE INTO THE TECHNICAL UNIT



1. RELEASE SILICONE FIXER.

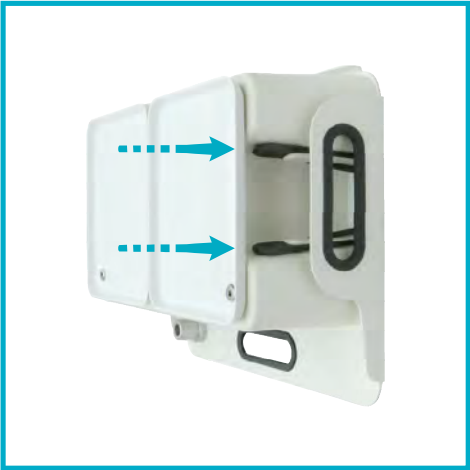

2. RELEASE SILICONE FIXER.

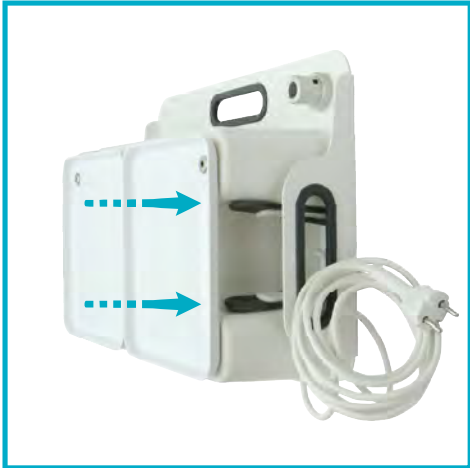

3. RELEASE SILICONE FIXER.



4. REMOVE DEVICE COVER.



5. BASE PLATE WITH FOUR RECESSES FOR THE DEVICE FEET.



6. PLACE THE FOOT CARE DEVICE DIRECTLY INTO THE FOUR RECESSES FOR THE DEVICE FEET.



9. PLUG POWER CABLE OF THE FOOT CARE DEVICE INTO THE MULTI-SOCKET OUTLET.



10. STOW THE POWER CABLE TO THE LEFT OF THE FOOT CARE DEVICE.




7. FOOT CARE DEVICE FULLY RESTS IN ITS POSITION.



8. STRAP THE FOOT CARE DEVICE TO THE BASE PLATE WITH THE FIXATION STRAP. IF NECESSARY, RE-OPEN THE FIXATION STRAP AND TIGHTEN IT.



11. INSERT THE BLACK CABLE BUSHING INTO THE FRONT RECESS OF THE DEVICE COVER.

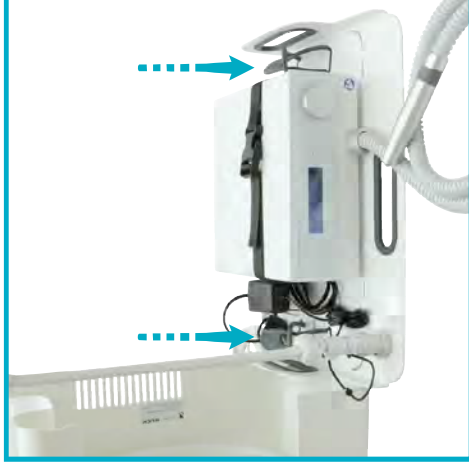


12. INTERLOCK THE SILICONE FIXERS INTO EACH OTHER.



13. INSERT THE DEVICE COVER INTO THE CIRCUMFERENTIAL GROOVE OF THE BASE PLATE.



14. ALIGN THE SUCTION HOSE OF THE FOOT CARE DEVICE.

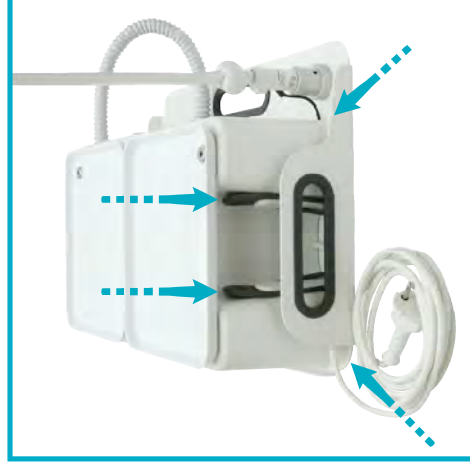


17. HANG THE SILICONE FIXERS INTO THE DEVICE COVER, ENGAGE THE DEVICE COVER IN THE BASE PLATE.



18. ROLL UP THE EXTENSION CORD OF THE MULTI-SOCKET OUTLET AND PLACE IT IN THE DESIGNATED SPACE.



15. INSERT THE HANDPIECE INTO THE HANDPIECE STORAGE LOCATION.

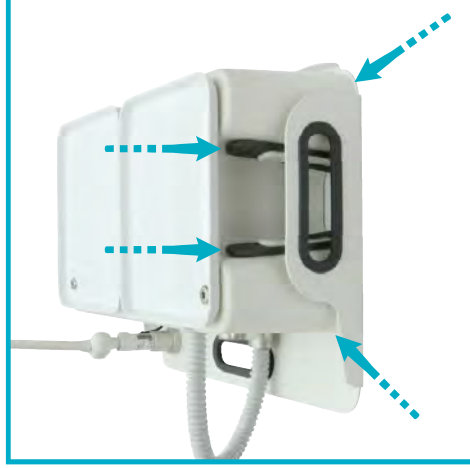

16. HANG THE SILICONE FIXERS INTO THE DEVICE COVER, ENGAGE THE DEVICE COVER IN THE CIRCUMFERENTIAL GROOVE OF THE BASE PLATE.



19. MOVE THE MAGNIFIER LAMP PAST THE SUCTION HOSE.

MOUNTING THE MAGNIFIER LAMP AND HANDPIECE HOLDER



1. REMOVE THE HOOD.



2. UNSCREW THE HEADLESS SCREW FROM THE LIGHT BRACKET.



3. PUT THE HANDPIECE HOLDER AND THEN THE O-RING OVER THE LAMP BASE.



4. TIGHTEN THE HEADLESS SCREWS OF THE LIGHT BRACKET. THE LIGHT SHOULD STILL BE ABLE TO TURN EASILY.



5. USE THE O-RING TO SECURE THE POWER CABLE (USE TWO IF NECESSARY).



6. SECURE THE POWER CABLE ON THE POWER ADAPTER AND PLUG IT INTO THE MULTI-SOCKET OUTLET.

CLEANING, DISINFECTING AND CARING FOR THE TECHNICAL UNIT

⚠ **Always unplug the main plugs before cleaning and/or disinfecting.**

⚠ **Do not spray cleaning and disinfecting agents into the multi-socket outlet.**

⚠ **Danger of damage due to incorrect care of artificial leather, plastics and wood surfaces!**

For protective cleaning of the surfaces, use a soft cotton cloth.

Caution: Cleaning cloths with plastic fibres can have an abrasive effect and damage the surface.

Take great care in ensuring that the cleaning utensils used are clean. If mineral contaminants such as grains of sand or dust are embedded in the cleaning agents, this can destroy the surface.

Do not use any concentrated disinfectants, solvents, cleaning agents or liquids that contain alcohol (> 50% alcohol component).

Do not use spray disinfectants.

Do not use any scouring agents, such as abrasive cleaners, cleaning sponges or similar items that may contain abrasive particles.

Residues of leaked fluids must be removed from the surface immediately with a soft cloth.

Over time, traces of use can appear on the surfaces. These are more or less noticeable depending on light incidence. They do not constitute a lessening of utility, but rather, as is the case with all surfaces over the long term due to the wear involved in the daily routine of the medical practice, they cannot be avoided.

Spots or stains caused by liquids that may corrode the surface (such as propolis, tea tree oil, cleaners that contain chlorine or similar substances) are categorically excluded from the warranty.

⚠ **Danger due to disease transmission!**

Insufficient disinfection can result in transmission of disease. Always comply with the requirements of the responsible national body for hygiene and disinfection. Use the RUCK® disinfection cloths – alcohol-free (art. no. 29428) – do not use any spray disinfectants. It must be possible to effectively evaporate surface disinfectants before the next treatment; allow surface disinfectants to thoroughly dry as specified by the manufacturer, but do not wipe surfaces dry.



1. REMOVE THE TECHNICAL UNIT FROM THE BAG.



2. REMOVE THE HOOD.



3. REMOVE ALL COMPONENTS FROM THE BASE PLATE.



4. THE TECHNICAL UNIT CAN NOW BE COMPLETELY CLEANED. IN ADDITION, THE SILICONE GRIP INSERTS MAY BE REMOVED FOR CLEANING.

WARRANTY

The product comes with a 2-year warranty. Normal wear of surfaces, castors, etc., and other moving parts is excluded from the warranty.

The warranty runs from the date of purchase when the product was first bought. The date of purchase has to be confirmed by the sales receipt. Within the scope of the warranty, HELLMUT RUCK GmbH will, at no charge, either repair or replace the defective products – at RUCK's discretion. The prerequisite is carriage-paid shipment of the defective product together with the purchase receipt. The local agent is responsible for claims made

outside of Germany. In case of doubt, please check with RUCK before returning the device.

The warranty becomes null and void if we find that the problem was caused by improper or incorrect installation or usage, by non-observance of the operating instructions, by external influence or as a result of an unauthorised repair or modification.

The warranty is limited to repair or replacement of the product. Further liability (particularly loss of revenue or earnings) is excluded. However, if it transpires that it is a problem that is not covered by the terms of the warranty or that the period of the warranty has elapsed, any costs for inspection and repair must be borne by the customer.

We reserve the right to make changes to and deviations from the technical design.



5. LUBRICATE THE MAGNIFIER LAMP SOCKET IN THE LIGHT BRACKET REGULARLY WITH RUCK MAINTENANCE FLUID (art. no. 10353).



6. LUBRICATE THE LOCKING NUTS ON THE ROTABLE INSTRUMENT TRAYS REGULARLY WITH RUCK MAINTENANCE FLUID (art. no. 10353).



7. READJUST LAMP HINGES. FOR THIS, REMOVE JOINT CAP AND SECURE WITH OPEN-END WRENCH (INCLUDED IN THE SCOPE OF DELIVERY OF THE LIGHTS).



TECHNICAL DATA

RUCK® MOBILE SYSTEM TECHNICAL UNIT BAG

Dimensions W/D/H: 47/18/35 cm

RUCK® MOBILE SYSTEM TECHNICAL UNIT RUCKSACK


Dimensions W/D/H: 35/22/48 cm





Our tip:
RUCK MOBILESYSTEM –
 The Mobile
 Footcare Handbook

This handbook answers all your questions and explains new treatment concepts from the business idea to hygienic instrument preparation.

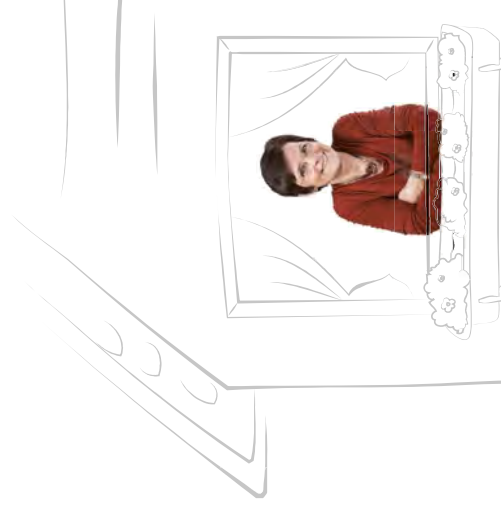

RUCK®

FÜR FUSS UND PFLEGE

HELLMUT RUCK GmbH

Daimlerstraße 23

D-75305 Neuenbürg, Germany

tel +49 (0)7082. 944 20

fax +49 (0)7082. 944 22 22

E-mail contact@hellmut-ruck.de

web www.hellmut-ruck.de