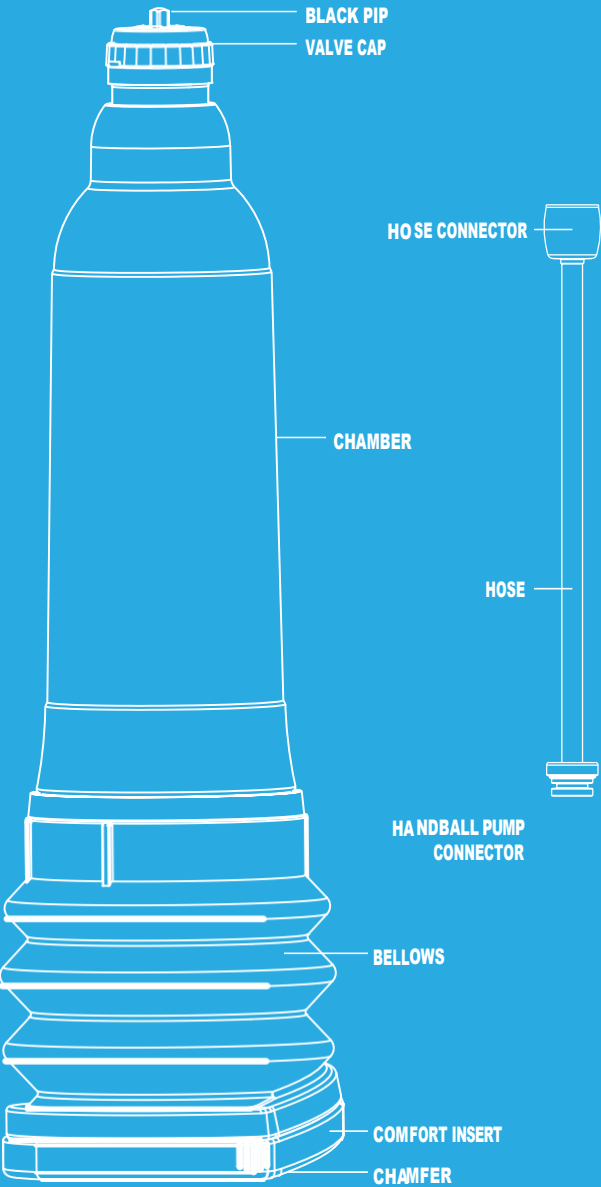
User Guide


Bruksanvisning

YOUR HYDROPUMP

VALVE POSITIONS

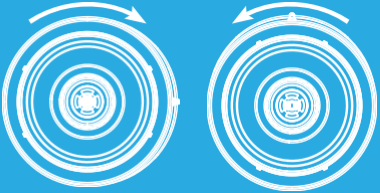LOCKED POSITION

UNLOCKED POSITION

OPEN POSITION

CLOSED POSITION

DEPRESSING THE VALVE CAP AGAINST THE SPRING RELEASES THE VACUUM

HANDBALL PUMP

HANDBALL VALVE CAP

What is a VaxAid hydropump?

The Vaxaid™ Hydropump is an external penile rigidity device using vacuum technology and an optional constriction ring to allow an erectile dysfunctional man to gain and maintain an erection suitable for sexual intercourse. Using VaxAid on a regular basis assists to maintain optimum penis health through regular exercise of the penis.

Why should you choose VaxAid?

Thanks to the fact that you can use Vaxaids vacuum technology in water, you exercise the blood vessels in the penis, increasing blood flow and increasing the chance of gaining an erection without the hydropump with each use. You can even get VaxAid hydropump on prescription in many parts of the world thanks to its outstanding ability to treat erectile dysfunction.

VaxAid does not claim to cure erectile dysfunction which may be caused by an underlying medical condition.

Precautions

The following precautionary measures are advised when using the device. As the product may cause sexual arousal with increased heart and respiration rates, persons should consult their doctor before using the product if you suffer any of the following: heart or lung disease, high blood pressure, stroke, epilepsy or convulsive disorder, genital-urinary disorder (e.g. kidney, bladder, testicular or penile problems), uncontrolled diabetes or if you take anti-coagulants, have sickle cell anaemia, bleeding disorders or leukaemia.

- This product is intended for use by a single user.
- Consult your doctor before using the Hydropump if you have reduced sensation in the genital area.
- You need to be in control of the product at all times. Avoid falling asleep while using the Hydropump or using the product whilst under the influence of alcohol or drugs.
- You must have full use of your hands to safely use the Hydropump, with the strength to operate the vacuum release mechanism.
- When using the VaxAid with a constriction ring in air, the penis and pubic area should be lubricated with the VaxAid proprietary water-soluble lubricant to prevent irritation.

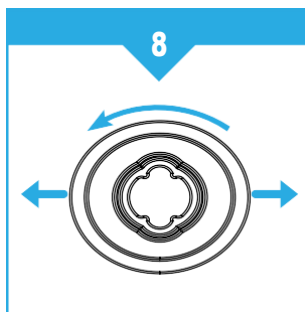
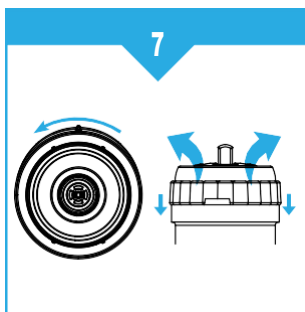
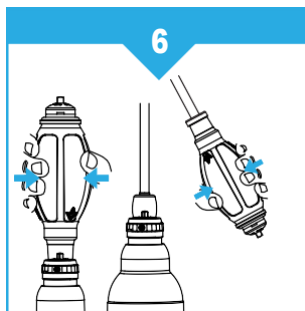
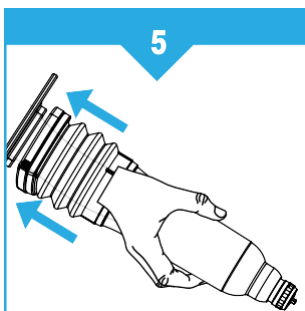
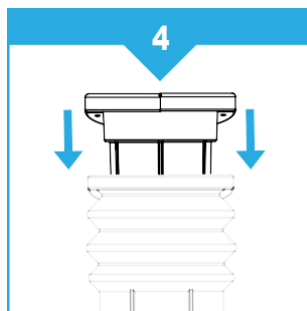
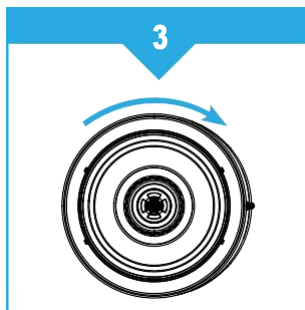
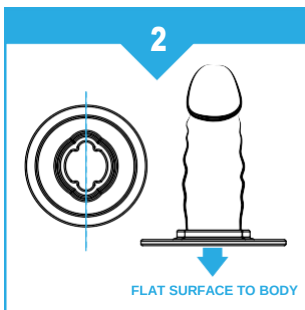
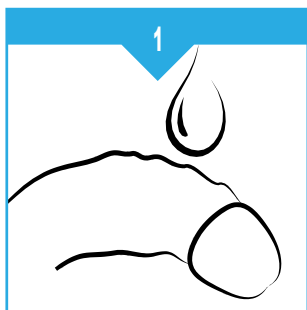
Recommended Exercise Regime

In order to obtain optimum benefit from the Hydropump when using it as a penile exerciser, and to avoid inappropriate use which may cause discomfort, you are strongly advised to follow the following exercise regime:

- Ensure your pubic area is reasonably well trimmed which will facilitate the vacuum seal against the pelvic seat.
- A relaxation period of approximately 3/5 minutes in a warm shower or bath is required to allow the testicles to soften, allowing them to remain free from the sealing area thereby avoiding any discomfort.
- Avoid vigorous over-pumping as this will not give the best results, and may cause discomfort and speckling spots or bruising.
- To benefit the most in erection hardness and optimal blood flow, it is best to remove the device every 2-3 minutes and allow the penis to retract. Massage the penis in the flaccid state, then re-apply the device and repeat the process. You should do this a maximum of 3 times in any 15 minute session.
- Do not use the Hydropump for more than 15 minutes in any period of 24 hours.
- When using a constriction ring care must be taken to ensure that the total time the ring is in place does not exceed 30 minutes.
- For further information and advice on getting the most benefit from your Hydropump, including instructional videos, please visit our website: www.vaxaid.com

USING YOUR HYDROPUMP IN AIR

The Hydropump is generally used in air as a precursor to sexual activity and will normally be used with a constriction ring to assist maintaining an erection suitable for sexual intercourse. When using a constriction ring care must be taken to ensure that the total time the ring is in place maintain an erection does not exceed 30 minutes.

How does the Hydropump work?

The Hydropump functions by creating a vacuum seal against the pubic area of the user enclosing the flaccid penis. When the pump is pushed downwards against the body, the bellows compress and water is ejected from the end of the pump via the open valve. When released, the bellows expand, the valve closes and creates partial vacuum inside the pump, this causes the penis to expand and fill with blood, creating an erection. Re-pumping increases the vacuum, and so increases the size of the erection.

The vacuum can also be increased by use of the handball pump, until optimum vacuum and a full erection is achieved. If connected directly, note that the fit between the handball pump and the valve is designed to be a non-permanent connection. For this reason the handball pump should be held in position whilst in use, and removed when optimum vacuum is achieved. If a more permanent connection is desired, please use the hose connector.

The user controls the valve by rotating the valve cap clockwise to lock the valve and anticlockwise to unlock the valve. When the valve is locked, the internal vacuum cannot be released, this position is used when pumping or using the handball pump. When unlocked the valve cap can be pressed inwards, opening the valve and allowing air or water back into the body of the pump, thereby releasing the vacuum.

The black pip on the top of the valve is only used to hold the valve closed when filling the Hydropump with water in the shower, at other times it is left free to move within the valve body.

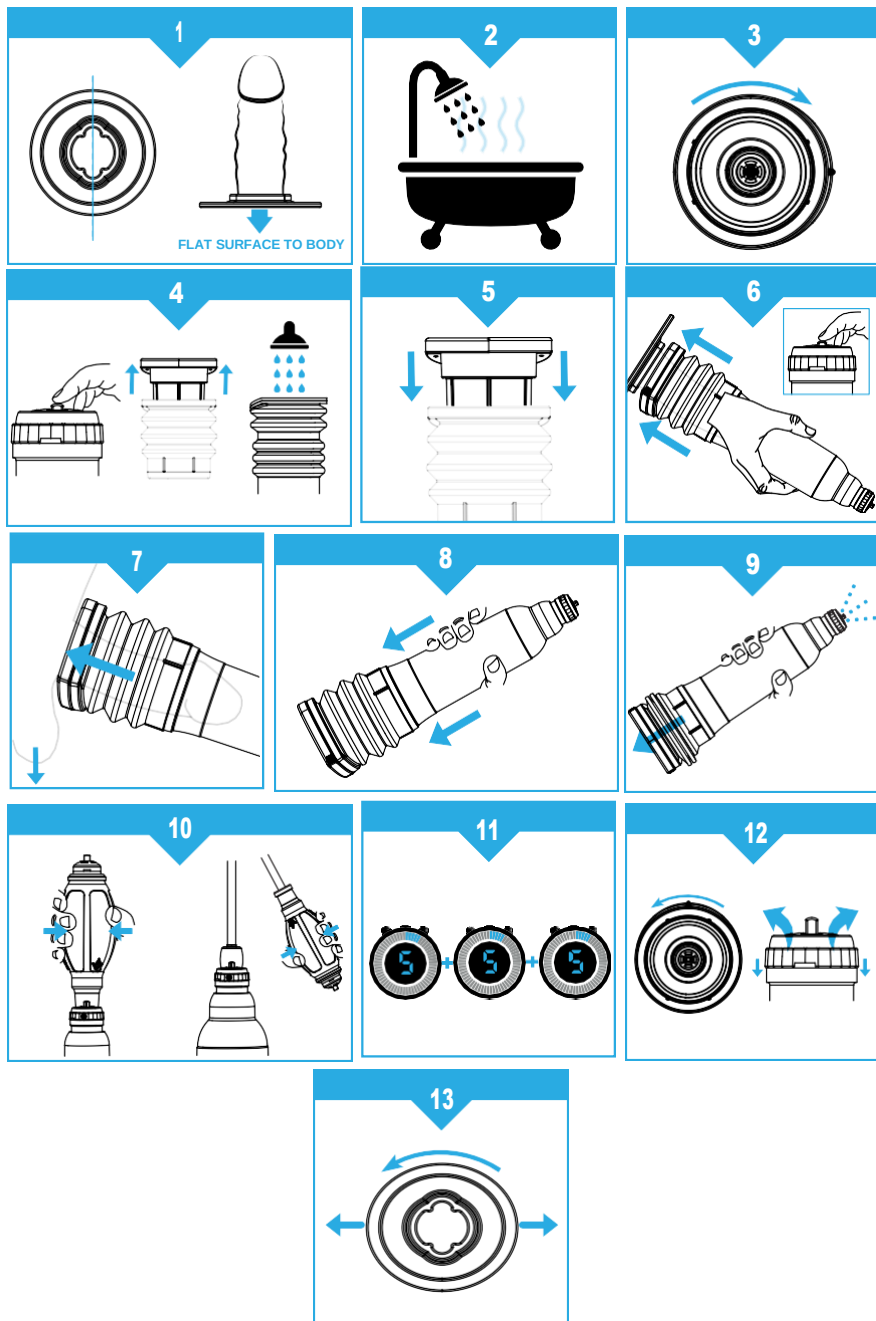
Using your Hydropump in Air

The Hydropump is generally used in air as a precursor to sexual activity and will normally be used with a constriction ring to assist maintaining an erection suitable for sexual intercourse. When using a constriction ring care must be taken to ensure that the total time the ring is in place maintain an erection does not exceed 30 minutes.

1. Apply lubricant to the shaft of the penis and genital area where the constriction ring will be fitted.
2. Select the correct size constriction ring and slide over the flaccid penis with the flat surface toward the body. Ensure the ring is aligned as shown. Fit it tightly against the pelvic seat, ensuring your testicles are well clear.
Important: Do not leave the constriction ring in position for more than 30 minutes.
3. Lock the valve by rotating the valve cap clockwise.
4. Ensure that the comfort insert is positioned tightly in position inside the pump bellows. The Hydropump body may be twisted in order to view the measuring gauge above or below the penis whilst in use.
5. Slide the Hydropump over the penis with the chamfer facing downwards, match up with the constriction ring and compress the bellows to form a tight seal.
6. Apply the handball directly to the valve cap or via the tube attachment and begin to pump until an erection is achieved.
7. When the desired erection is achieved, remove the Hydropump by rotating the valve cap anti-clockwise to unlock the valve, depress the valve cap inwards to release the vacuum. Slide the Hydropump off your penis, leaving the constriction ring in place to maintain your erection.
8. Remove the constriction ring by stretching it apart and rotating the ring through 90° - this will allow your penis to start to become flaccid and the ring may be comfortably removed."

USING YOUR HYDROPUMP IN WATER - SHOWER

The Hydropump is generally used in water for penile exercise, either with or without a constriction ring. When using a constriction ring care must be taken to ensure that the total time the ring is in place to maintain an erection does not exceed 30 minutes.

Using your Hydropump in Water

The Hydropump is generally used in water for penile exercise, either with or without a constriction ring. When using a constriction ring care must be taken to ensure that the total time the ring is in place to maintain an erection does not exceed 30 minutes.

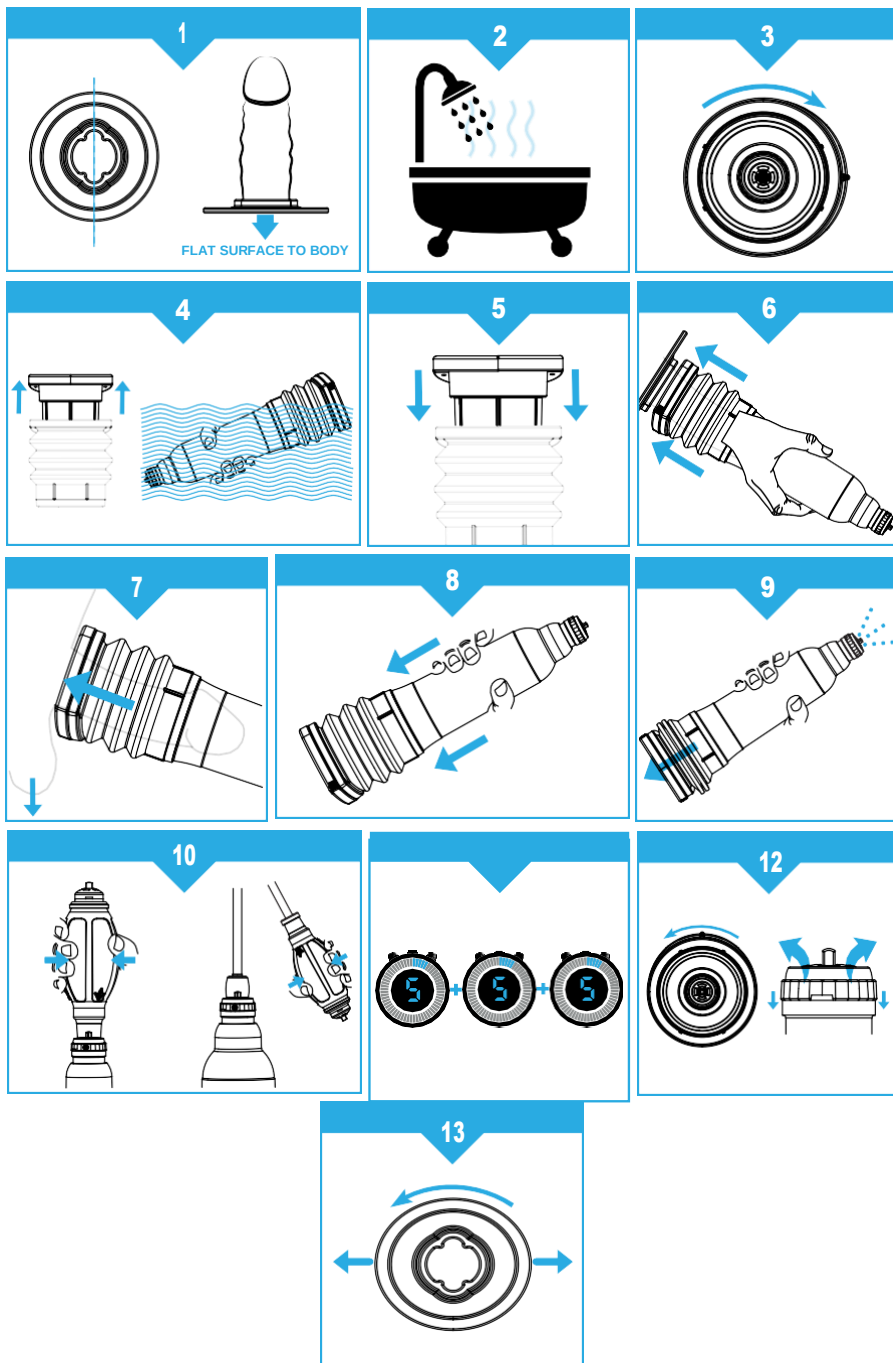
Using your Hydropump in the shower

If you have purchased the optional shower strap, you can slide the connector onto the Hydropump leaving your hands free to wash.

1. If the use of a constriction ring is desired, the ring is best fitted before entering the shower – follow the instructions as shown in the section above Using your Hydropump in Air. **Important: Do not leave the constriction ring in position for more than 30 minutes.**
2. Relax in the Shower for a few minutes.
3. Lock the valve by rotating the valve cap clockwise.
4. Push and hold the black pip into the closed position with your finger, remove the comfort insert (to prevent trapped air) then invert the Hydropump under the shower head until full of water.
5. Replace the comfort insert.
6. Whilst holding the black pip in the closed position, place the Hydropump over your flaccid penis tight up to your pelvic seat. The chamfer can be positioned facing either upwards or downwards depending on your level of comfort.
7. Make a seal against your body, ensuring that the positioning is comfortable and that your testicles are well clear.
8. With the Hydropump angled slightly upwards commence pumping.
9. Pump slowly and evenly until the bellows remain depressed, indicating that optimal vacuum has been achieved.
10. To increase the vacuum further and for more control, use the hand ball pump either with the hose or connected direct to the pump. Please note, the fit between the handball pump and the Hydropump valve is designed to be a non permanent connection. For this reason the handball should be held in position whilst in use, and removed when optimum vacuum is achieved. If a more permanent connection is desired, please use the hose connector.
- 11. Ensure you follow the recommended instructions.**
12. To remove the Hydropump, unlock the valve by rotating the valve cap anti-clockwise and release the vacuum by depressing the valve cap inwards to activate the quick-release mechanism and slide the Hydropump off your penis.
13. Remove the constriction ring by stretching it apart and rotating the ring through 90° - this will allow your penis to start to become flaccid and the ring may be comfortably removed."

USING YOUR HYDROPUMP IN WATER - BATH

The Hydropump is generally used in water for penile exercise, either with or without a constriction ring. When using a constriction ring care must be taken to ensure that the total time the ring is in place to maintain an erection does not exceed 30 minutes.

Using your Hydropump in the bath

1. If the use of a constriction ring is desired, the ring is best fitted before entering the shower – follow the instructions as shown in the section above Using your Hydropump in Air. **Important: Do not leave the constriction ring in position for more than 30 minutes.**
2. Relax in the bathtub for a few minutes.
3. Lock the valve by rotating the valve cap clockwise.
4. Remove the comfort insert (to prevent trapped air) and fill the Hydropump with bathwater.
5. Replace the comfort insert.
6. Place the Hydropump over your flaccid penis tight up to your pelvic seat. The chamfer can be positioned facing either upwards or downwards depending on your level of comfort.
7. Make a seal against your body, ensuring that the positioning is comfortable and that your testicles are well clear.
8. With the Hydropump angled slightly upwards commence pumping.
9. Pump slowly and evenly until the bellows remain depressed, indicating that optimal vacuum has been achieved.
10. To increase the vacuum further and for more control, use the hand ball pump either with the hose or connected direct to the pump. Please note, the fit between the handball pump and the Hydropump valve is designed to be a non permanent connection. For this reason the handball should be held in position whilst in use, and removed when optimum vacuum is achieved. If a more permanent connection is desired, please use the hose connector.
- 11. Ensure you follow the recommended instructions.**
12. To remove the Hydropump, unlock the valve by rotating the valve cap anti-clockwise and release the vacuum by depressing the valve cap inwards to activate the quick-release mechanism and slide the Hydropump off your penis.
13. Remove the constriction ring by stretching it apart and rotating the ring through 90° - this will allow your penis to start to become flaccid and the ring may be comfortably removed."

Cleaning and Maintenance

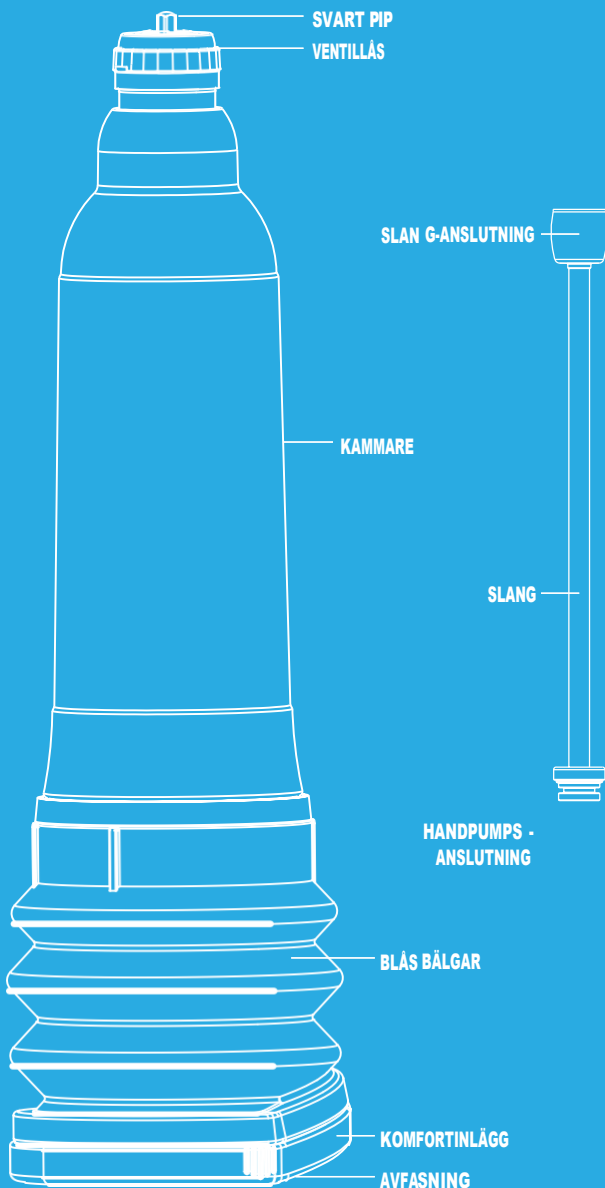
Wash out your Hydropump with warm soapy water and rinse thoroughly. Allow your Hydropump to dry and store in a dry place away from direct sunlight. If you have purchased the Vaxaid cleaning kit, please follow the instructions provided. To maintain the hygiene of the Hydropump, it is advisable to remove the comfort pad for cleaning and storage. This prevents trapped water remaining between the insert and the bellows, which can also adversely affect the life of the product.

Warranty



The Vaxaid Hydropump is designed to provide many years of trouble free use. It is covered by a two year warranty against defective parts or poor workmanship. Full details of the warranty can be seen on our website: www.vaxaid.com

In the unlikely event of a warranty claim, Vaxaid Ltd reserves the right to request the return of the Hydropump or provision of photographic evidence of failure, damage etc.

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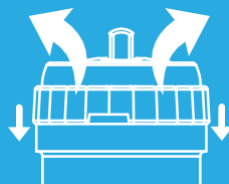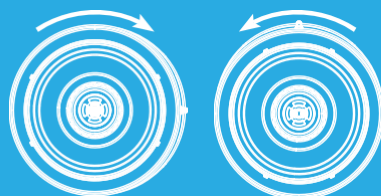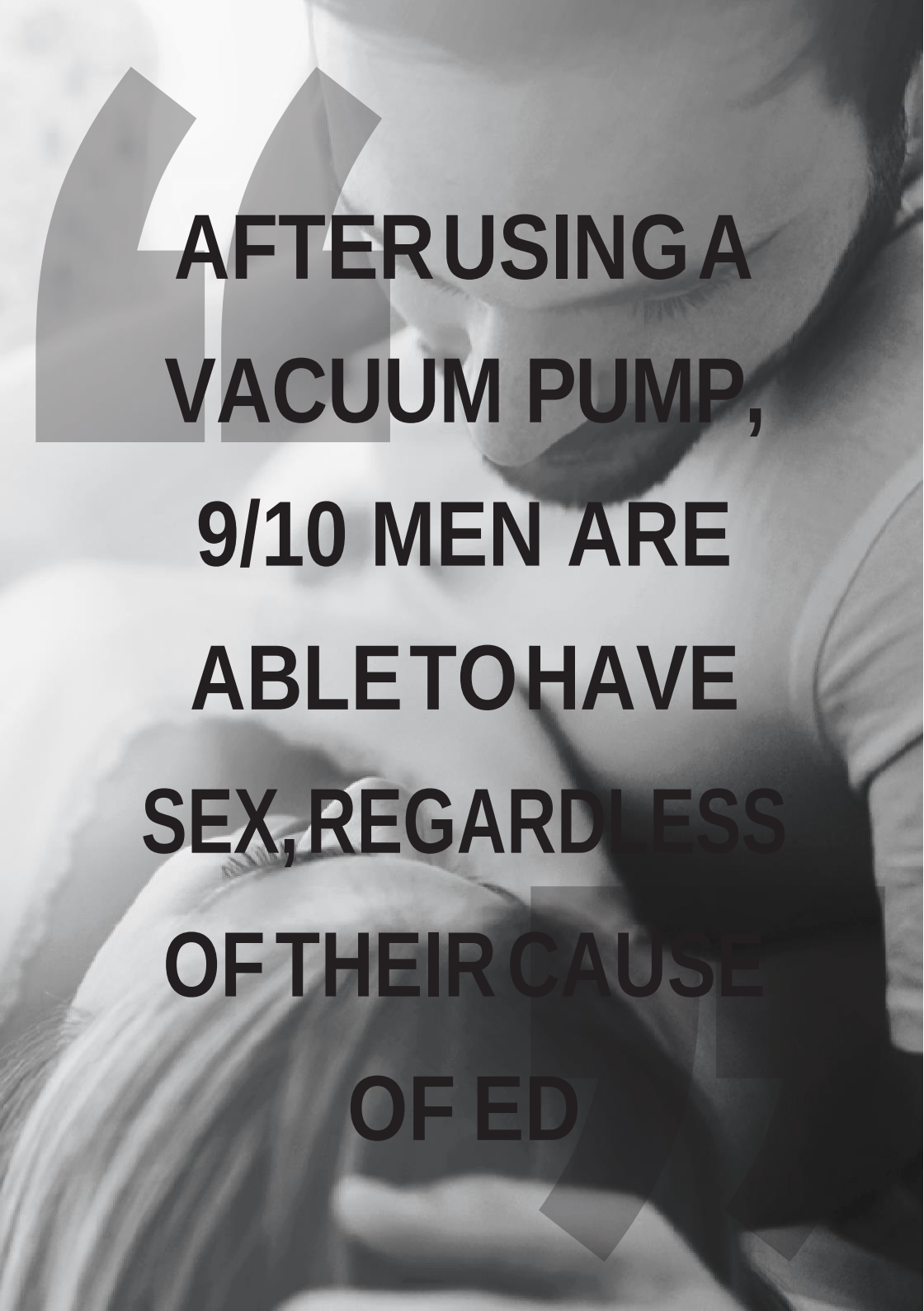
**AFTER USING A
VACUUM PUMP,
9/10 MEN ARE
ABLE TO HAVE
SEX, REGARDLESS
OF THEIR CAUSE
OF ED**

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AFFIX SECURITY STICKER

Warranty Registration

All VaxAid© Hydropumps are supported with a two year warranty.

You can easily register your Hydropump online at: www.vaxaid.com

Alternatively, send details of your name, address & VaxAid© serial number to the address below.

Unit 1, Horbury Jct Ind Est. Calder Vale Rd, Horbury, Wakefield, WF4 5ER, UK

VaxAid© is a Worldwide Registered Trademark.

See website for more information – www.vaxaid.com

© Worldwide copyright protection - VaxAid Ltd

Manufactured in the UK by VaxAid Ltd

Unit 1, Horbury Jct Ind Est. Calder Vale Rd, Horbury, Wakefield, WF4 5ER, UK

+44 (0)1924 288094 www.vaxaid.com

Swedish distributor: Jewa Sweden AB – info@jewasweden.se
Elementvägen 2, 437 36, Lindome, Sverige, Phone: +31 49 50 80

Design Registration: EU 002111069/2, USA 29/450986
Patent nos: UK 2418686 EU 1799162, USA 7651463, AU 2005278999

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Tillverkad i Storbritannien av VaxAid Ltd
Svensk distributör: Jewa Sweden AB – info@jewasweden.se